COLDISEPT

NANO SILVER

Throat spray

EN

INSTRUCTION FOR USE

A medical device in the form of a spray for supportive treatment of the throat inflammation caused by bacterial and viral infections.

COLDISEPT nanoSILVER Throat Spray creates a protective layer on throat mucosa, protecting it against physical, chemical and biological factors.

COLDISEPT nanoSILVER Throat spray helps eliminate constant discomfort through moistening a dried mucosa of the throat, thereby noticeably reducing the symptoms of inflammation such as pain and burning. The product creates a layer of gel which protects the mucous membranes of the mouth and the throat from bacteria and viruses.

The aqueous solution of sodium, potassium, calcium and magnesium mineral salts contained in the product cleanses the throat, reduces the swelling of the throat and nasal mucosa and supports the regeneration of irritated mucous membranes.

The product contains silver nanoparticles that limit the growth of bacteria. Peppermint oil helps clear the nasal passages, relieves cough and improves breathing comfort.

COMPOSITION

Purified water, nanocolloid silver, *Chondrus crispus* (carrageenan) extract, sorbitol, glycerin, mineral salts (potassium dihydrogen phosphate, potassium chloride, sodium chloride, calcium chloride hexahydrate, magnesium chloride hexahydrate), peppermint oil, sodium benzoate, citric acid.

INDICATIONS

COLDISEPT nanoSILVER Throat spray effectively supports the treatment of:

- inflammation of the mucous membranes of the mouth and the throat;
- inflammation of tonsils;
- condition after removal of tonsils.

SIDE EFFECTS

Side effects of COLDISEPT nanoSILVER Throat Spray are not known, however, in particularly sensitive persons, allergic symptoms cannot be excluded. If an allergic reaction occurs, stop using the product.

ADDITIONAL INFORMATION AND PRECAUTIONS

Do not use the product if you are allergic to any of the ingredients.

Do not use the product if you are allergic to metallic silver.

Prevent accidental contact with the eyes.

Stop using the product in case of any alarming symptoms.

Do not use after the expiry date!

Pregnant women and children under 6 years of age may use the product only under medical supervision.

The product can be used over long periods of time without the risk of addiction, side effects or complications.

USAGE AND DOSAGE

Use the product 3-4 times a day, in accordance with individual needs.

Press the dispenser several times before the first use.

Place the applicator as deeply as possible in the mouth and point it towards the throat.

Apply 3-4 doses (applicator pump presses).

STORAGE

Store at a temperature under 25 °C, in a dry place, out of reach of children.

Packaging 20 ml

MEANING OF SYMBOLS:

	Batch number		Consult instruction for use		Manufacturer
	Use-by date		Date of manufacture		Temperature limits

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**Manufactured in the EU for:
Alpen Pharma AG,
Bern, Switzerland**

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